

# Set & Forget


Simplistic.  
Easy to use.  
**Set once. Ready to dose.**



Set & Forget, as the name suggests, allows the user to set-up the automed delivery device and adapter once using the App, and then continue dosing while leaving the app behind.

Data is stored on the device and can be synchronized with the App after dosing is completed.

# Getting Started

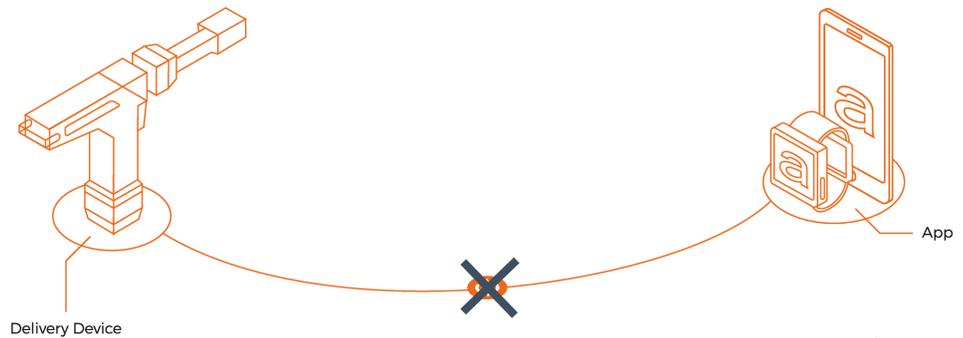

## Android

Connect the Android device (running on Android 7, 8, or 9) to the automed system through a hotspot.

1. Open Settings on the Android device
2. Click Mobile Hotspot
3. Click Name and change it to AutoMed
4. Change the Password for the hotspot to 12345678
5. Turn the Mobile Hotspot On

## iOS

Connect the iPhone to the automed system through a Personal Hotspot on the iPhone.

1. Open Setting on the iPhone.
2. Click General
3. Click About
4. Click on Name and change it to AutoMed
5. Return to the Settings page and click Personal Hotspot
6. Change the Password for the hotspot to 12345678
7. Turn the Personal Hotspot On

## Enable Set & Forget

### Connecting the automed device to the Essentials App.

1. Download the automed Essentials app in the App Store
2. Open the automed Essentials App
3. Clip the adapter onto the automed device
4. Insert the battery into the device
  - a. Once battery is inserted into the device, wait for the Blue Status Bar at the top of the phone saying you have at least 1 connection (iOS)
5. Click the Connect button in the app

### Configure a Blank adapter

1. Once the device is connected to the app, a popup will appear at the bottom of the app to Select the Dose Method.
2. Select Manual
3. Type in the Dose Value
  - a. This is the amount in ml/cc that will be delivered with the automed device
  - b. Click Done
4. The dose will appear on the Essentials app screen
5. Click Disconnect
6. The device's LED light will turn Red for a second and then turn back to Green to indicate it is ready for dosing

A user is now able to dose animals with the automed device without being connected to the app. Once an adapter is configured, the app does not need to be reconnected. Clip the adapter onto the device, insert the battery, and the device is ready to dose the set amount with that configured adapter.